

Instructions For Use: Similac[®] Nipple & Ring Attachment to Similac[®] Bottles

1. Place bottle upright on level surface and carefully remove cap if present.
2. To open Similac Infant Nipple, peel paper from clear film while holding the nipple and ring with opposite hand through the clear film to prevent touch contamination.
3. Grasp bottle with free hand.
4. Without removing the clear film, place the nipple on the bottle. Using extended fingers only, (Figure 1), screw the nipple ring clockwise until ring stops rotating.
5. Do not tighten further.
6. Remove clear film from nipple.

CAUTION: DO NOT OVERTIGHTEN

Overtightening nipple ring to bottle, normally associated with grasping nipple ring with a technique other than extended fingers (Figure 2), can result in thread skipping, ring pop-off, leakage or improper venting.



Figure 1



Figure 2